

Accurate Results for your pet. *As they should be.*

A different kind of meter...

The CentriVet™ GK Blood Glucose & Ketone Monitoring System is the only dual meter device that **measures both blood glucose and ketone** levels, giving you accurate results in seconds.

Specifically calibrated for cats and dogs, it has never been easier to take control of your pet's health!



- ✓ Affordable tests for everyday monitoring
- ✓ Monitor glucose & ketone levels in seconds
- ✓ Easy-to-use
- ✓ Suitable for testing both cats and dogs
- ✓ Portable, handheld device
- ✓ Stores up to 450 tests
- ✓ 3000 test battery life
- ✓ Designed for convenience and safety




Technology used by vets, now available for home use

- ✓ Two tests in one meter for glucose and ketone testing
- ✓ Clinically proven accuracy
- ✓ Species-specific calibration chips for testing both dogs and cats




Canine Glucose Accuracy Study
CentriVet™ Blood Glucose & Ketone Monitoring System

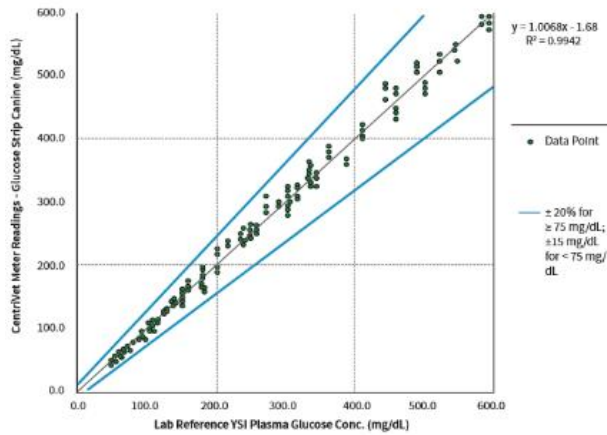

System Accuracy Results for Glucose Concentrations ≥ 75 mg/dL and < 75 mg/dL	
Within $\pm 15\%$ of ± 15 mg/dL	Within $\pm 20\%$ of ± 15 mg/dL
356/360 (98.9%)	360/360 (100%)

Feline Glucose Accuracy Study
CentriVet™ Blood Glucose & Ketone Monitoring System

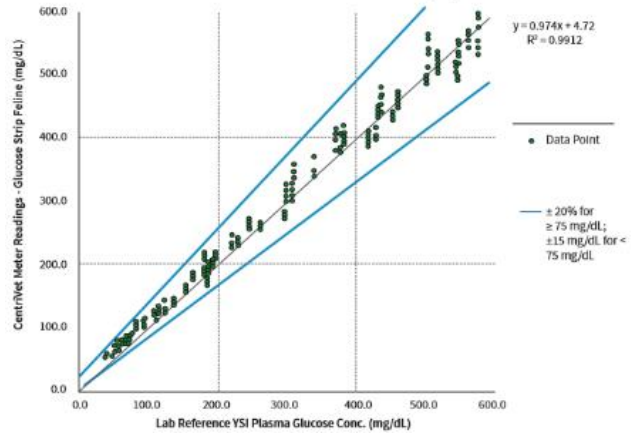

System Accuracy Results for Glucose Concentrations ≥ 75 mg/dL and < 75 mg/dL	
Within $\pm 15\%$ of ± 15 mg/dL	Within $\pm 20\%$ of ± 15 mg/dL
357/360 (99.2%)	359/360 (99.7%)

Full Specifications

Feature	Details
Sample Size	Glucose: 0.8 μ L β -Ketone: 1.2 μ L
Test Time	Glucose: 5 seconds β -Ketone: 10 seconds
Test Range	Glucose: 0.6 – 33.3 mmol/L (10 – 600 mg/dL) β -Ketone: 0.0 – 8.0 mmol/L
HCT Range	20-60% for both Glucose and β -Ketone
Memory Storage	450 blood glucose and ketone records with result type, date, and time
Test Averaging	7, 14, 30, 60, and 90- day averaging for glucose, before and after meal markers for glucose testing
Battery Life	3,000 tests
Operating Temperature Range	5 - 45°C (41-113°F)
Operating Humidity Range	10 – 90%
Automatic Shutoff	2 minutes of inactivity
Power Source	2 CR 2032 3.0V coin cell batteries

Ordering Information

Product	Contents	Item Code
CentriVet™ GK Blood Glucose & Ketone Meter Kit	Meter, Batteries, Carrying Case, User's Manual, Reference Materials, Warranty Cards	GK114-13C
CentriVet™ Blood Glucose Test Strips	50 Strips/vial	GK134-12CB
CentriVet™ Glucose Control Solution	2 x 2 mL bottles	GK124-13C
CentriVet™ Blood Ketone Test Strips	25 Strips/vial	GK134-10CB
CentriVet™ Ketone Control Solution	2 x 2 mL bottles	GK124-12C

CentriVet's products are distributed exclusively in the UK by Rapid Labs Ltd.

www.rapidlabs.co.uk

Tel: +44 (0) 1206 250484

Email: info@rapidlabs.co.uk



Hall Farm Business Centre, Church Lane, Little Bentley, Essex CO7 8SD.